


# DISPENSER

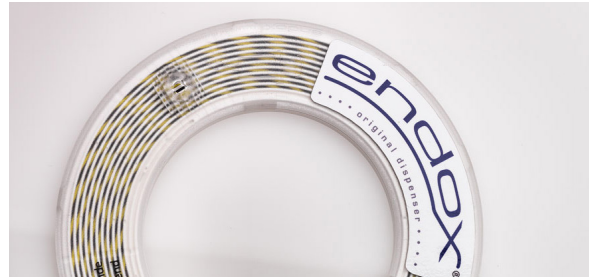

The Dispenser is assembled and delivered aseptically packed. Upon request with your logo as a sticker or as a pad printing.

It consists of two halves rotatable relative to each other. For defined and safe use a stopper is provided before the outlet aperture, which can also be used at the same time for attaching a clipper for suspending, e. g. around the neck.

The left hand positions the thumb here. On the back side there are wells which allow to mete the guide wire with the right hand or to rewind it again. The gradation in the outlet aperture enables the packer to introduce the guide wire according to the length of the wire.

## The gradation amounts to:

4,6 m  
4,0 m  
3,3 m  
2,6 m  
1,8 m  
1,0 m

